# Charge and Care Clean UV Drybox

Brugervejledning  
Bedienungsanleitung  
Instruction Manual  
Manuel d'utilisation  
Felhasználói kézikönyv  
Manuale di istruzioni  
Gebruiksaanwijzing  
Instrukcja obsługi  
instruções de utilização

#### **DK Advarsler:**

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#### **EN Warnings:**

Children under the age of 8 years, people with reduced eyesight or sense of touch can only use this product under supervision. USB Cable can be a strangulation risk. Do not use the Charge and Care in a humid environment. The surface of the Charge and Care may become hot during use. Do not store the Charge and Care in hot places such as a hot car or behind glass in the sun. UV-C light is used to clean the hearing aid. This light should turn off immediately when opening the lid of the Charge and Care. If for some reason, you are able to see the UV-C light, disconnect the power supply and return the product to the store.

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## Specifications

- Product name: Charge and Care
- Model number: CAC-70S
- Compatible with AudioNova receiver in canal hearing aids, including AudioNova B-R, AudioNova B-RT, AudioNova DX, USB cable ④ and USB Power Adaptor. ⑤

## Instruction manual

The Charge and Care cleans and dries your hearing aids by a cycle of UV-C light and hot air flow. It can also charge your hearing aids. The Charge and Care kills over 99% of tested bacteria and fungus (plate count ISO 4833-1).

### Required power adaptor

To make good use of all the functionalities of the Charge and Care, it is necessary to use the supplied power adaptor or an equivalent USB certified medical power supply with the correct specifications. The delivered voltage should be between 4,75V and 5,25V and the current should be 1000mA, otherwise the Charge and Care may not work properly or malfunction.

Adapter input	AC 100-240V~ 50/60Hz 0,6A
Adapter output	5,0V – 1000mA

### Device setup

Please insert one end of the USB cable into the USB-C port of the Charge and Care. Insert the other end of the cable into the USB port of the power adapter. After connecting the Charge and Care, the Power LED indicator ① will light up directly and remain green. The Charge and Care is now ready for use.

### Charging and cleaning

Clean the hearing aids according to the manufacturer's cleaning instruction to remove all visible residue. Before placing the hearing aids in the cradles, make sure the hearing aid is dry. The LED on the hearing aid will light up when the hearing aid is positioned correctly in the Charge and Care. Only when this is done correctly the charging process will start, so check carefully before closing the lid. The LED's on the hearing aids will turn green when the hearing aid is fully charged. Don't move the Charge and Care during use and make sure the air vents stay free of blockages.

Close the lid and press the start button ②. The blinking LED indicator ② on the start button will turn white and the Charge and Care will begin its 2,5 hours cleaning cycle. The cleaning program starts with a ten minute UV-C cleaning cycle and continues with a hot airflow of 30°C. The cleaning program ends after a second cycle of UV-C cleaning.

If the Charge and Care is opened during the UV-C cleaning cycle, the UV-C light will turn off immediately for safety reasons. The cleaning and charging process will be on hold for 5 minutes. When the LED indicator  on the start button is blinking red, close the lid and the Charge and Care will continue the cleaning and charging process directly. When the lid is not closed within 5 minutes, all cycles will end.

#### Completion mode

To turn off the Charge and Care after completion it is needed to disconnect the power adapter from the power supply. The Power LED indicator  will stay on until disconnected from the power adapter. To interrupt the cleaning and charging process, open the lid for over 5 minutes or disconnect the power source. Let the hearing aids cool down for at least 5 minutes before taking them out. Take out the hearing aids with an upward motion to remove them from the Charge and Care .

#### Charging mode

It is possible to charge the hearing aid without using the cleaning cycle. Connect the Charge and Care to the power supply. Insert the (single) hearing aid in the charging cradle and close the lid without pressing the start button. The LED indicator on the start button  will blink during the entire charging session. Depending on the type of hearing aid, it will take approximately 3 hours to fully charge the hearing aid. When the LED lights on the hearing aids turn green they are fully charged.

#### Care of the Charge and Care

Before cleaning the Charge and Care, first disconnect it from the power supply. We recommend to clean it weekly by using moist wipes. Please avoid direct contact with water.

#### Power supply

If a power adapter is used with less than 1000mA charging will be slower, however the cleaning process will be completed. If a power adapter with more than 1000mA is used, the Charge and Care will limit the power supply to 1000mA to protect the electronics.

If the power adapter is not connected correctly or the effectively supplied current is below 500mA, the Charge and Care will stop working.

#### Failure mode

When the hearing aid is connected correctly to the Charge and Care, but the hearing aid battery is completely empty the LED on the hearing aid will not light up directly, this might take some time.

When a wrong type of hearing aid is placed in the charging cradle and the Charge and Care makes electrical contact, no LED at the hearing aids will light up and the hearing aid will not be charged.

When the power adapter is broken, the power LED indicator  will not light up.

When the UV-C light, fan or heating elements are broken, the LED indicator on the start button  will not respond when being pressed.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer.

**TECHNICAL SPECIFICATIONS**

Model	CAC-705
Input	4,75 - 5,25 V
UV-C light	Wavelength 254 nm
UV-C time	2 x 10 minutes per cycle
Heating time	130 minutes per cycle
Working conditions	Indoor Use (IP21)
Operation altitude range	0 m up to 2000 meter
Operation temperature	5°C - 40°C (41° to 104° F)
Operation humidity	10% to 70%
Operation environment	82kPa - 106 kPa
Supply voltage fluctuations	Not more than 10%
Adapter input	AC100- 240V~50/60Hz 0,6A
Adapter output	5,0V – 1000mA
Adapter model	GA-0501000V

**TRANSPORT AND STORAGE CONDITIONS**

Temperature	-10°C to 50°C (14° to 122° F)
Humidity if not in use	0% to 70%
Atmospheric pressure	50kPa to 106kPa
Keep dry	
Expected shelf life	24 months
Dispose	At waste disposal sites intended for electronic waste.
Pollution degree	Class II

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**WAKE UP INTERNATIONAL B.V.**